ESCAPE

MATTRESS®

MOBILITY COMPANY

Solo - Fold - Compact - Brancard/Stretcher

Manual - Manuel - Gebrauchsanleitung - Handleiding

English

Thank you for your purchase! Your Organisation has made the best choice in investing in our evacuation products. Optimize your investment in Escape-Mattress® by taking advantage of training possibilities. We advise your employees to conduct training in the use of the Escape-Mattress®, on a yearly basis. Please contact us for further information on training possibilities.

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Installation and training instructions

Manufacturer recommended use of the Escape-Mattress® Fold, Compact, Solo and Stretcher.

This Escape-Mattress® is used to transport people who have been injured, or are disabled, for those who cannot manage to cross a horizontal route or who needs to be transported down the stairs in, a horizontal and safe position.

General

This manual is intended to provide the user with the information required to use the Escape-Mattress® in a safe and correct manner. It contains technical data, information about maintenance, parts and use.

The Escape-Mattress® must be used by trained persons only.

When lifting, four people should always use all eight handles located along the sides of the Escape-Mattress®.

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Caution:

- The user must not make any alterations to the Escape-Mattress®. Escape Mobility Company declines any responsibility for the consequences of incorrect use or damage caused by unauthorised changes.
- Always protect yourself against illnesses, of which can be transferred by blood or other bodily fluids.
- Only use the Escape-Mattress® as described within the manual.
- Two or more able-bodied rescue workers are required to use the Escape-Mattress® for an evacuation via the stairs.
- Always respect the maximum permissible load as mentioned in this manual on page 1.
- When transporting persons who are heavier than yourself, it is important that you are aware of the restrictions and possibilities. Trained persons can estimate such an evacuation.
- Use the **Escape-Mattress® Stretcher** as per the make and model of stretcher mentioned on the identification label. For the purpose of the safety restraint, make use of the loops (refer to page 3, point 7).

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Technical specifications/ List of parts:

1. Headrest
2. Body flap
3. Towing strap (folded)
4. Foot bag
5. Safety belt with buckle
6. Fixing strap for the head
7. For the purpose of the safety restraint, loop

Caution! Training is only allowed to be given by persons who have been certified by Escape Mobility Company.

The location of the Escape-Mattress®

We advise you to install the Escape-Mattress® in a strategic (and preferably central) place.

Cleaning the Escape-Mattress®

The material can be disinfected with diluted disinfectants (such as, e.g., chlorine preparations or quaternary ammonium) in accordance with the disinfectants instructions. However, discolourations may occur. Do NOT bring the material into contact with solvents or solvent-containing detergents or disinfectants (such as, e.g., alcohol).

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The material is resistant to blood, urine, acids and alkaline. Iodine and alkaline leave discolourations in the material.

Initial use of the Escape-Mattress®:

1. **Model Fold:** Click open the buckle which keeps the Escape-Mattress® together and unfold the mattress. Position the headrest, if this has not already been done.

Model Compact: Remove the Escape-Mattress® from the bag and fasten the headrest to the mattress.

Model Solo: Unzip the case which contains the Escape-Mattress®.

Remove the headrest from its compartment and position it onto the mattress.

Model Stretcher: Remove the Escape-Mattress® from the stretcher and unfasten the body flaps at the underside of the Escape-Mattress®. Lay the Escape-Mattress® flat on the floor and fasten the headrest (*refer picture 1*).

Do not use the Escape-Mattress® without the headrest.

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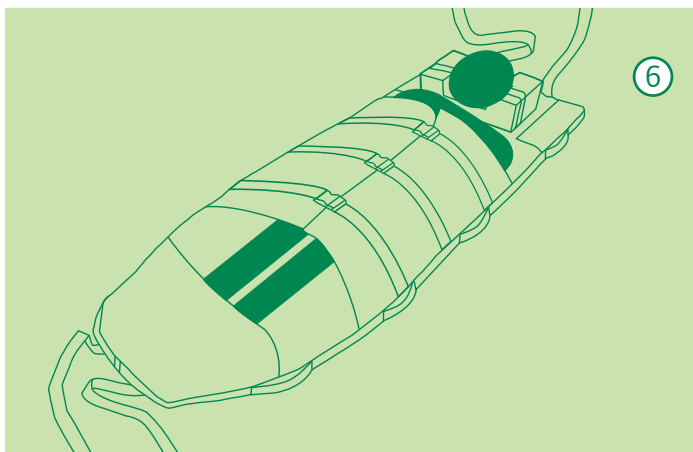
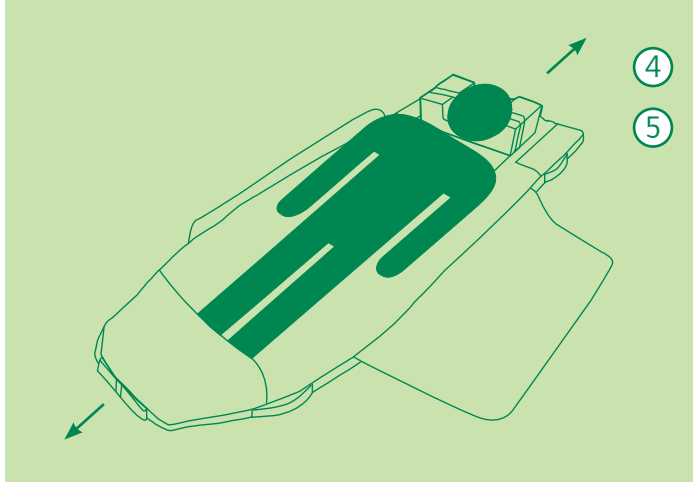
2. Make sure that the foot bag faces the direction of the route to be travelled.
3. Click open the buckles of the belts and open the de body flaps (refer picture 3).
4. Free the towing straps at the head end and the foot end by releasing the velcro strip (refer picture 4).
5. Assist the evacuee onto the Escape-Mattress®. The head needs to be positioned on the headrest with the feet secured in the foot bag (refer picture 5). Check that the headrest is in the correct position.
To change the position of the headrest remove the headrest from the velcro and re-adjust to the desired position.
Close the fixing strap of the headrest, in order to hold the head in place.
6. Place the body flaps around the evacuee and close the safety belts (these must 'click'). Always make sure that the arms of the evacuee are under the body flaps (refer picture 6).
7. Pass the ends of the safety belts under the closed belts so that they do not dangle freely.

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
8. Evacuate the person. This requires at least two persons: one at the head end and one at the foot end. Pull the evacuee in the direction of the staircase, feet first, using the long towing straps (*refer picture 8*). If you raise the foot end slightly, the friction resistance decreases.
9. In order to negotiate bends, it may be easier to:
- use the shorter strap of the towing straps (Model Fold, Solo and Compact).
 - use the extra handles, half way along the towing straps (Model Stretcher).
10. The person at the foot end now steps onto the stairs. When negotiating the bend, it helps to raise the foot end slightly. (This also goes for a turn on a landing). Let the person who is at the foot end rest the mattress against their hip / upper leg, so that the speed of descent can be properly controlled. This also helps the person at the head end (*refer picture 10*). Evacuate down the stairs at a calm pace. The person at the head end must be in sync with the movement. In this way it is also possible to continue to

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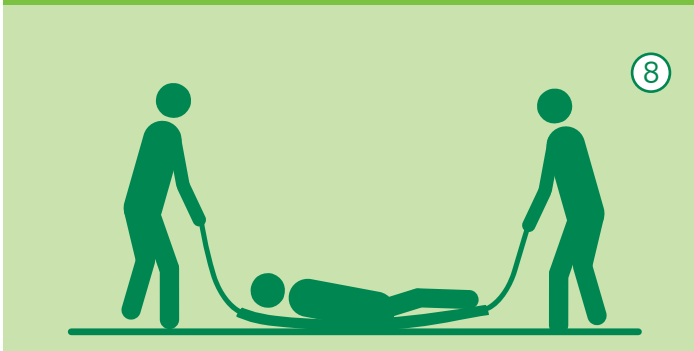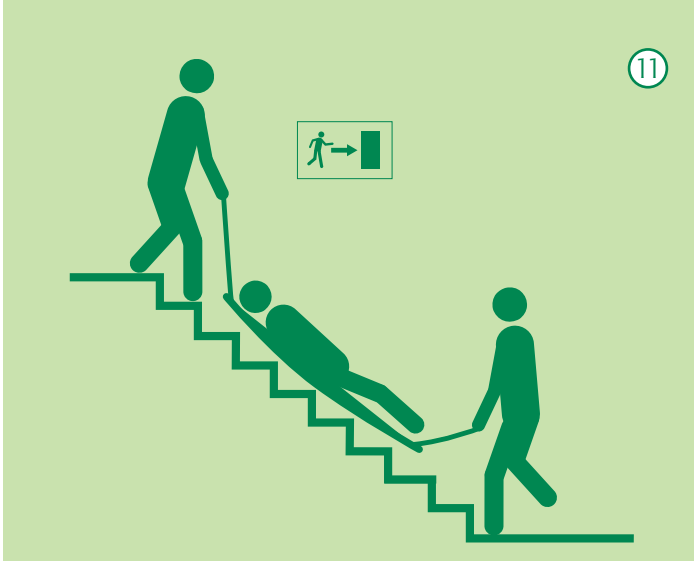
walk upstraight maintaining a good body position.

11. At the bottom of the stairs the person at the foot end must pay attention to the straps by continuing to walk forward (refer picture 11). The person at the head end must slightly increase their speed on the final step, in order to make the transition easier.
12. Tow the evacuee to a safe location or the assembly point.
13. Unfasten the body flaps and help the evacuee get off the Escape-Mattress® (refer picture 13).

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Storing the Escape-Mattress® after use:

1. Inspect the mattress for damage and/or wear. Faults are allowed to be repaired by the supplier only. If necessary the mattress can be cleaned as described on page 3.
2. Fold the towing straps and store them behind the velcro strips provided for that purpose.
3. Stretcher mattress: fold the body flaps towards the underside and fasten them together with the velcro strip provided.
4. Place the Escape-Mattress® back into the case (if applicable).
5. Return the headrest in the storage compartment provided (if applicable).

Interim changes in technical specifications, printing and composing errors excepted.

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Escape Mobility Company Escape-Mattress® Solo
Escape Mobility Company Escape-Mattress® Fold 55/70
Escape Mobility Company Escape-Mattress® Compact
Escape Mobility Company Escape-Mattress® Brancard

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- NEN-EN 12182
- NEN-EN 1865

Gedaan te Landgraaf,

A handwritten signature in blue ink, which appears to read "H. L. G. van Leeuwen", is written over a horizontal blue line.

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Certifies under its own responsibility that the following products with ICG registration number NL-CA002-2012-23816:

Escape Mobility Company Escape-Mattress® Solo
Escape Mobility Company Escape-Mattress® Fold 55/70
Escape Mobility Company Escape-Mattress® Compact
Escape Mobility Company Escape-Mattress® Stretcher

to which this certificate relates, are in compliance with the requirements of the European Directive concerning medical devices 93/42/EEC, being classified as class I in accordance with appendix IX.

And comply with the following (harmonised) standards, which have been applied entirely or in part:

- NEN-EN 12182
- NEN-EN 1865

Drawn up in Landgraaf,

A handwritten signature in blue ink, which appears to read "H.L.G. van Leeuwen", is written over a horizontal blue line.

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