

**User manual Escape-Mattress®
Bariatric**

1	Remove the Escape-Mattress® Bariatric from the storage bag and place it on the ground.	
2	Unfold the Escape-Mattress® Bariatric and make sure the safety belts are facing towards you.	
3	Place the adaptable head rest on the Velcro straps at the head end. Attention: do not use the Escape-Mattress® without the headrest.	
4	Click open the buckles of the belts and open the de body flaps on both sides.	
4	Place the evacue (e) on the Escape-Mattress® Bariatric. Do this with enough assistants. Make sure the head is placed on the headrest and fix the head with the fixation straps.	
6	Place the feet of the evacuee in the foot bag.	
7	Place the body flaps around the evacuee and close the safety belts (these must 'click'). Always make sure that the arms of the evacuee are under the body flaps.	
8	Pass the ends of the safety belts under the closed belts so that they do not dangle freely.	

9	<p>Evacuate the person. This requires at least 4-6 persons. Use the extra drag straps that can be hooked onto the mattress. Pull the evacuee in the direction of the staircase, feet first, using the long towing straps. If you raise the foot end slightly, the friction resistance decreases.</p>	 
10	<p>In order to negotiate bends, it may be easier to:</p> <ul style="list-style-type: none"> • use the shorter strap of the towing straps. • use the handgrips on the sides of the mattress. 	 
11	<p>Now let 2 people stand at the foot end and let them step up the stairs. By lifting the foot end one can make the turn to the stairs easier (also the turn on a landing platform). Have the persons standing at the foot end support the mattress by resting it against their hip / thigh, so that you have good control over the speed of the descent. By doing this they also relieve the people at the head end.</p>	
12	<p>The other 2 to 4 people take place at the head end. You can now attach 2 extra drag straps at the head end. This allows better control of the weight of the evacuee.</p>	
13	<p>Evacuate down the stairs at a calm pace. The person at the head end must be in sync with the movement. In this way it is also possible to continue to walk up straight maintaining a good body position.</p>	
14	<p>Tow the evacuee to a safe location or the assembly point.</p>	
15	<p>Unfasten the safety straps and open the body flaps and help the evacuee get off the Escape-Mattress® Bariatric.</p>	
16	<p>Inspect the Escape-Mattress® Bariatric for damage and/or wear.</p>	
17	<p>If necessary, clean with water and disinfectant soap.</p>	

Replace the Escape-Mattress® Bariatric at the designated location.

Attention:

The Escape-Mattress® Bariatric may be used by trained persons only.

Maximum carrying capacity of the Escape-Mattress® Bariatric is 300 kg.

Faults or changes are allowed to be repaired/done by the manufacturer only.

Save this manual.